initial^{IQ}

GC Initial IQ
Lustre Pastes NF
V-Shades

EN Technical Manual

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FR Manuel Technique

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ES Manual técnico

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Master all your challenges

JUNE 2018

GC

EN	DE	FR	IT	
Table of Contents	Inhaltsverzeichnis	Table des matières	Indice	Page
Intended Use and Shelf Life	Bestimmungsgemäßer Gebrauch und Haltbarkeit	Utilisation et péremption	Campo di impiego e durata dei prodotti	3-4
Description of the Lustre Pastes NF	Beschreibung der Lustre Pastes NF	Description des Lustre Pastes NF	Descrizione delle Lustre Pastes NF	5-7
The liquids of the Lustre Pastes NF Set	Die Flüssigkeiten des Lustre Pastes NF Set	Les liquides du coffret Lustre Paste NF	I liquidi del set di Lustre Paste NF	8-9
Lustre Pastes NF Application	Lustre Pastes NF Anwendung	Application Lustre Pastes NF	Applicazione delle Lustre Pastes NF	
Basic Colour Combination Chart	Kombinationstabelle Basis-Farben	Tableau de combinaisons des teintes Basic	Schema generale delle combinazioni cromatiche	10
Basic Step by Step Instruction	Basis-Farben Step by Step Anleitung	Instruction étape par étape	Istruzioni di base passaggio per passaggio	11
Advanced Colour Combination Chart	Individuelle Farb-Kombinationstabelle	Tableau de combinaison des teintes chromatiques	Schema avanzato delle combinazioni cromatiche	12
Advanced Step by Step Instruction	Individuelle Farben Step by Step Anleitung	Instruction étape par étape	Istruzioni avanzate passaggio per passaggio	13
Finishing	Finishing	Finition	Rifinitura	14
Firing Chart	Brenntabelle	Tableau des températures	Grafico delle cotture	15
Tips and Tricks	Tipps und Tricks	Trucs et astuces	Consigli e accorgimenti	16
Facilities of the Lustre Pastes NF	Möglichkeiten mit Lustre Pastes NF	Possibilités offertes avec les Lustre Pastes NF	Esempi di applicazioni delle Lustre Pastes NF	17
Cut Back Technique modified with GC Initial CL-F	Cut Back Technik mit GC Initial CL-F modifiziert	Technique Cut Back modifiée avant GC Initial CL-F	Tecnica di Cut Back modificata con GC Initial CL-F	18-19
FAQ	FAQ	FAQ	Domande più frequenti	20-22

GC Initial IQ Lustre Pastes NF V-Shades

EN Intended Use and Shelf Life

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EN Colour deepness and lifelike translucency

GC Initial IQ Lustre Pastes NF (LP NF) are 3-dimensional ceramic stains developed to bring colour deepness and lifelike translucency for the complete GC Initial ceramic range. Thanks to their "New Formula" you can now really bring your crowns and bridges to life with this dedicated Lustre Pastes NF Set, with an optimal match to both "low" and "high" CTE ceramics. Available in a ready to use consistency, the Lustre Pastes NF are based on fine ceramic particles and can be applied in a thicker layer where they will exhibit unsurpassed vitality and a very natural glaze to your restorations.

Features

- Just one dedicated kit for all different indications for the complete GC Initial ceramic range.
- The GC Initial IQ Lustre Pastes NF are based on fine ceramic particles, enabling a thicker application compared to conventional stains.
- Improved and unique paste medium, it has a fine thixotropic property for easy and precise applications.
- Available in a ready to use consistency (1 x Neutral, 1 x Value, 4 x Body Shades, 8 x Enamel Effect Shades).
- The high fluorescence of the Lustre Pastes ensure the look of natural teeth.

Indications

The Lustre Pastes NF are a one for all solution for the complete Initial line from GC. Its possible to use the Lustre Pastes NF in a wide CTE – Range between 6,9 and 13,3 in order to work with nearly all types of ceramic. These Lustre Pastes NF represent a modern alternative for conventional staining and glazing, as well as for the colour, brightness, grey value and surface texture corrections of all of your ceramic crowns and bridges. Due to the unique properties it's just a question of minutes to individualize the occlusal surface of your crown and bridge work, to paint on white bands, decalcifications, hair lines and crack lines or to characterize artificial porcelain teeth.

Keep in mind: What you paint (on) is what you get (after firing)

GC Initial IQ Lustre Pastes Set NF

Contents:

1x Neutral, 1 x Value, 4 x Body Shades, 8 x Enamel effect shades, Diluting Liquid, Refresh Liquid, 1 mixing dish, 2 brushes, Picture IFU

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● EN Lustre Paste 'Diluting Liquid':

The 'Diluting Liquid' is a special liquid to change the flowability of the Lustre Pastes. However, it is recommended to obtain a gel/paste like consistency of the Lustre Pastes for best results.

Attention:

- If the Lustre Pastes are too much diluted, the result is a reduction in the gloss of the paste after the firing; hence, the desired result may not be achieved in one glaze firing.
- When needed, dilute the desired amounts of the pastes in a separate plate with Diluting Liquid, never dilute directly in the jars.

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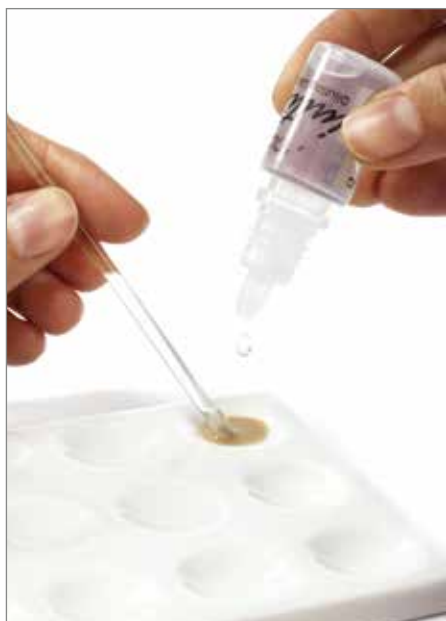
● **EN** Lustre Paste 'Refreshing Liquid':

The 'Refreshing Liquid' is used to preserve the thixotropic property of the pastes. This liquid is used to recover dried out pastes to an optimum, gel like consistency without changing the physical properties. The Refresh Liquid can be used both in the jars and mixed on a separate plate.

Important: Please shake the Refresh Liquid well before use!

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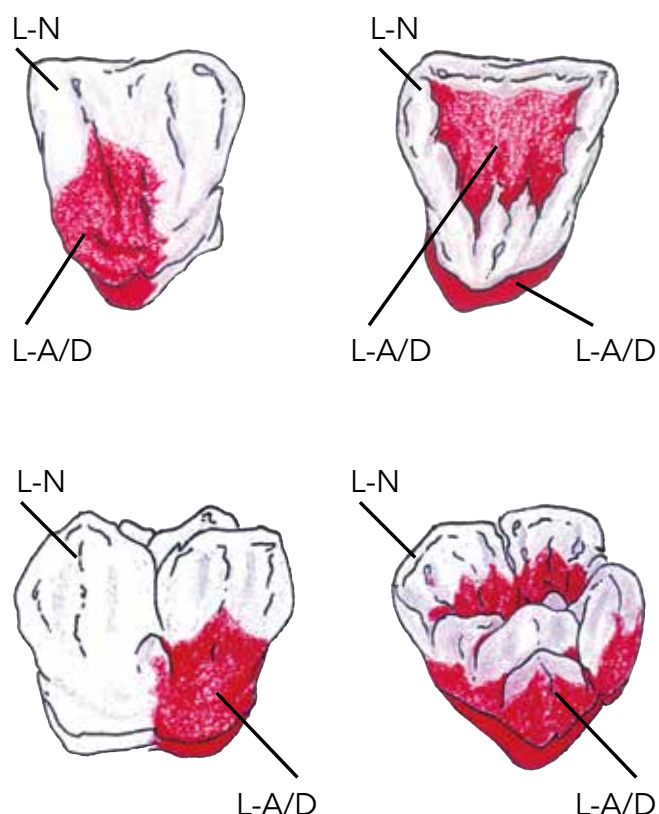
EN Important Information: Please always stir up material well in the jars before use.
Before applying the Lustre Coating, the ceramic restoration is shaped and contoured using adjusted diamond burs. After prepolishing with a ceramic polisher the finished crowns & bridges are sand-blasted with 50 microns Alu-Oxid at 1,5 bar pressure.

The Lustre Paste Neutral is coated on the entire surface of the sandblasted restoration. This coating is clearly thicker than what we know from a normal glaze firing.

Note: These Lustre Pastes may not come in contact with water.

The requested Vitapan Classic® Shade is created by a more or less intensive application of the Lustre Pastes "Body A - D". This process, which only takes 2 minutes, is everything which is necessary to create Vitapan Classic® shades.

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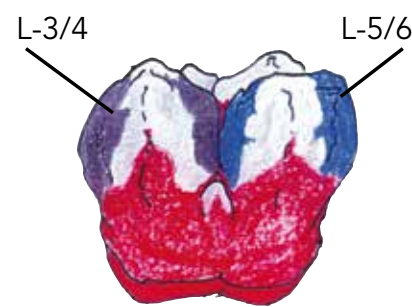
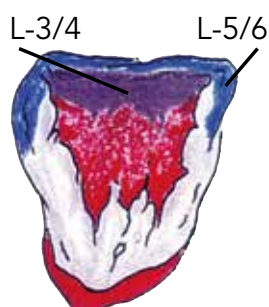
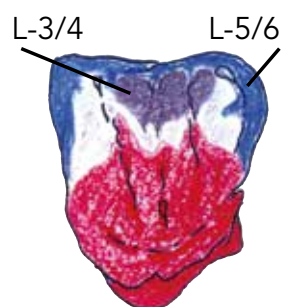
EN To build up a high aesthetic solution for the patient use our advanced step by step instruction:
 In order to achieve the higher grey value of the incisal area with darker teeth (A3.5; A4; C3; C4), 2 grey Effect Shades can be used: L-3 (Light Grey) and L-4 (Dark Grey). For eventual further individualisation, you can use the whitish Lustre Pastes NF: L-1 (Vanilla) and L-2 (White) and L-V (Value) for brightening the restorations.
 2 bluish Effect Lustre Pastes NF: L-5 (Light Blue) and L-6 (Dark Blue) for a bluish opalescent appearance in the incisal area.
 Greenish Effect Lustre Paste NF: L-8 (Olive) for a greenish effect. Can be mixed with all Lustre Pastes NF.

EN In order to achieve an even higher „Transparency“ the 2 greyish Effect Lustre Pastes NF L-3/L-4 can be used as well.
 All GC Initial MC INvivo Stains can be used to achieve even more possibilities of individualization.

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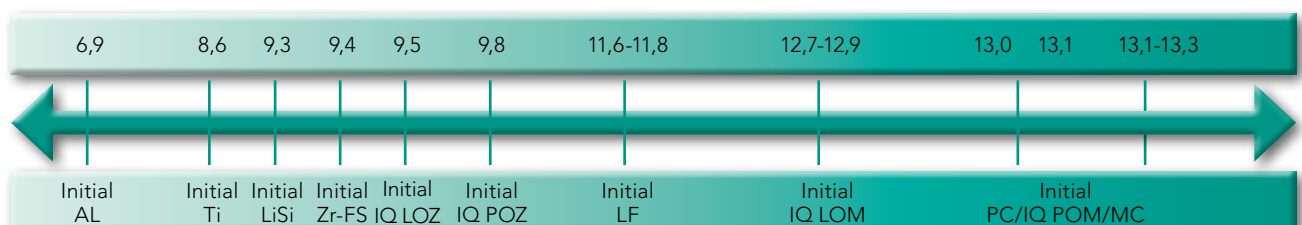
EN Additional changing's in colour, transparency and brightness of crown and bridge work can be done smoothly with the Lustre Pastes. The unique consistency ensures homogenous modifications and appearance.

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EN Make your corrections with the effect colours of the Lustre Paste Set.

Increase brightness > LP Value
Increase colour saturation > L A-D
Increase grey value > L 3/4

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EN Use the Lustre Pastes to inhibit the depth effects and the transparency of your crown and bridge work build in layering technique. Sandblast your restoration with 50 micron Aluminumoxide, damp it and then paint it on with Lustre Paste L-N.

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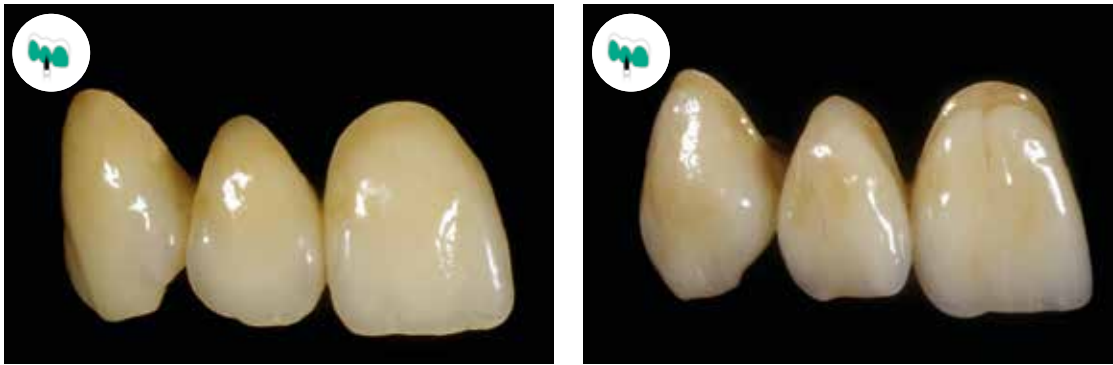
EN The requested surface texture/smoothness of the Lustre Pastes layer can be obtained by soft vibrating or condensing the restoration.

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- GC Initial Metal Ceramic Systems (MC, LF, Ti)
- GC Initial IQ Press-over-Metal Concept
- GC Initial IQ One-Body Layering Concepts
- GC Initial Full Ceramic Systems (AL, Zr-FS)
- GC Initial IQ Press-over-Zircon Concept
- GC Initial Pressable Ceramic Systems (PC, LF)
- GC Initial LiSi

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EN The restorations layered or pressed in the One Body method can be easily reduced with appropriated abrasive wheels and discs. The finishing can be done with different ceramics of the Initial Family.

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EN The high transparency of Initial IQ Press Over/Layering Over ceramics requires an additional chromatisation of the remaining dentin body before you start the finishing. Therefore apply the required Lustre Pastes NF colours, or even the Initial INVIVO MC/Zr Stains and fix them in a brief firing on the dentin body (750°C – 1 minute holding time). For the finishing use the required firing parameters fitting to the used ceramic.

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EN The coloured, overburned dentin body is first fully covered with CL-F (MC or Zr-FS) and then layered with Opal/Incisal masses on the final shape of the tooth. (See also Initial Manual)

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EN The finished with the Initial Lustre Pastes NF glaze fired crowns are indistinguishable from high end, polychrome layered restorations.

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DE FAQ

FR FAQ

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EN **Basic Rule:** What you paint (on) is what you get (after firing)

Q1. How can I dilute (adapt the consistency of) the Lustre Pastes NF?

Use the Lustre Pastes NF Diluting Liquid to adapt the consistency. Do NOT use water to dilute. LP may not come in contact with water (see page 9).

Q2. What can I do when the consistency of the Lustre Pastes NF is too dry?

Use the Refresh Liquid to recover the Lustre Pastes NF in case of dryness (see page 9).

Q3. How do we change the texture of the Lustre Pastes NF?

The requested surface texture/smoothness of the Lustre Pastes NF layer can be changed by soft vibration or condensing the applied pastes.

Q4. I do not have a shiny appearance after firing the Lustre Pastes NF, what is the cause?

Check the firing temperatures, increase the final temperature. When the Lustre Pastes NF are heavily diluted or applied in a too thin layer the glossy effect is tempered, so apply a more thick layer/dilute less the pastes.

Q5. Can I apply more than one layer of Lustre Pastes NF?

Yes, you can do further firings, keeping the same firing temperatures.

Q6. Can I mix the Lustre Pastes NF with other Initial powders?

You can use the GC Initial Stain powders to intensify the Lustre Pastes NF. Mix the required amount of stain powder with the LP NF Diluting Liquid and mix it with the required LP NF or directly mix the stain powders with f.i. the Lustre Paste Neutral (L-N).

Q7. I have finished my veneered crown (MC/LF) and I want to use the Lustre Pastes NF without losing the surface texture of the prepared crown.

You can easily dilute the LP to have the optimal consistency in order not to lose the surface texture and still have a nicely coloured and shiny crown, just apply in a thin layer.

Q8. Are the Lustre Pastes NF sufficient abrasion resistant?

Yes, the LP NF are a special mixture of different types of low fusing ceramic particles, stains & glaze mixture offering colour deepness and lifelike translucency.

Q9. I have to change the colour, brightness or the grey value of a crown made with the layering technique. What can I do?

Use the Lustre Pastes NF as shown on page 16 to modify your crown and bridge work.

Q10. May I use the Lustre Pastes NF for AL in the same way I use it for other materials?

No, you should apply and fire the Lustre Pastes NF in 2 thin layers.

Q11. After firing the colour of my work is spotted?

The Body Colour needs to be massaged into the Lustre Paste Neutral (L-N) on your crown or should be mixed equally on an external mixing board in advance. To disperse the colour pigments it is necessary to condensate the Lustre Pastes NF before the glaze firing by soft vibration.

Q12. The colour doesn't fit with the Vita®Classic Shade guide?

Different material thicknesses cause different colour effects. Therefore, it may be necessary in individual cases, to mix an A-Body Colour with a B-Body or even with L8 (Olive). A visual check is always possible.

Q13. The glaze firing is clearly, spotted or cracked?

The Lustre Pastes NF came partly or completely in contact with water.

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GC NORDIC AB
GC AUSTRIA GmbH
GC AUSTRALASIA DENTAL PTY LTD