Prior to use, carefully read the instructions for use. **EN**

G-aenial™ Universal Injectable

UNIVERSAL LIGHT-CURED RADIOPAQUE INJECTABLE COMPOSITE

For use only by a dental professional in the recommended indications.

INDICATIONS FOR USE




1. Direct restorative for Class I, II, III, IV and V cavities
2. Fissure sealant
3. Sealing hypersensitive areas
4. Retention of (in)direct aesthetic restorations, temporary crown & bridge, defect margins when in enamel
5. Blocking out undercuts
6. Liner or base
7. Core Build-up

CONTRAINDICATIONS

1. Direct pulp capping.
2. Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.

DIRECTIONS FOR USE

1. Shade Selection
Select shade from 16 shades of XBW, BW, A1, A2, A3, A3.5, A4, B1, B2, CV, CVD, AO1, AO2, AO3, JE, AE.
A, B, C, D shades are based on Vita® Shade.
*Vita® is a registered trademark of Vita Zahnfabrik, Bad Säckingen, Germany.
2. Cavity Preparation
Prepare cavity using standard techniques. Dry by gently blowing with oil free air. Rubber dam is recommended to isolate the prepared tooth from contamination with saliva, blood or sulcus fluid.
For pulp capping, use calcium hydroxide.
3. Bonding Treatment
For bonding G-aenial Universal Injectable to enamel and/or dentin, use a light-cured bonding system such as G-Premio BOND, G-aenial Bond or G-BOND. Follow manufacturer's instructions.
4. Placement of G-aenial Universal Injectable

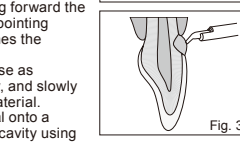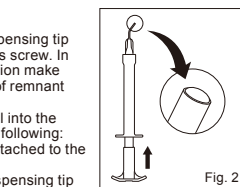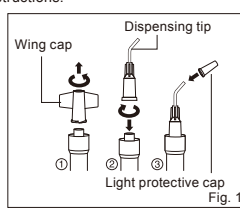Take care not to attach the dispensing tip too tightly. This may damage its screw. In order to ensure a tight connection make sure that the threads are free of remnant material.
Prior to dispensing the material into the cavity, make sure to check the following:
a. Dispensing tip is securely attached to the syringe.
b. Any possible air from the dispensing tip is removed by gently pushing forward the syringe plunger with the tip pointing upwards until material reaches the opening of the tip (Fig. 2).

5. Place the dispensing tip as close as possible to the prepared cavity and slowly push the plunger to extrude material. Alternatively, dispense material onto a mixing pad and transfer to the cavity using a suitable instrument.
Note:
a. If plugged, remove the dispensing tip and extrude a small amount of material directly from the syringe.
b. Minimise exposure to ambient light. Ambient light can shorten the manipulation time.
c. After use, immediately remove and dispose of the dispensing tip and lightly close the syringe with the wing cap.

2) Dispensing from a Unitip
Insert the G-aenial Universal Injectable Unitip into the Unitip APPLIER II or equivalent. Remove the cap and extrude material directly into the prepared cavity. Use steady pressure (Fig. 3). Maintain pressure on the applicator handle while removing the Unitip APPLIER II and Unitip from the mouth. This prevents the Unitip from coming loose from the applicator.
Note:
The material may be hard to extrude immediately after removing from cold storage. Prior to use, leave to stand for a few minutes at room temperature.

Clinical Hints

In order to inject effectively, use the surface tension of the material to ensure uniformity across the entire surface of the restoration during build up. Once the required amount has been injected, release the pressure on the plunger and withdraw the syringe in a direction perpendicular to the surface. This will allow the material to separate from the dispensing tip and provides a smooth surface over the restoration.

Light-curing

Light-cure G-aenial Universal Injectable using a light curing unit. Keep light guide as close as possible to the surface. Refer to the following chart for Irradiation Time and Effective Depth of Cure.

	Irradiation Time and Effective Depth of Cure (ISO 4049)	
	10 sec. (High power LED) (>1200mW/cm ²)	20 sec. (Halogen/LED) (>700 mW/cm ²)
A1, A2, A3, B1, B2, JE, AE	2.5mm	2.0mm
XBW, BW, A3.5, A4, AO1, AO2, CV, CVD	2.0mm	1.5mm
AO3	1.5mm	

Note:
a. The material should be placed and light-cured in layers. For maximum layer thickness, please consult above table.
b. Lower light intensity may cause insufficient curing or discoloration of the material.

Finishing and Polishing

Finish and polish using standard techniques.

SHADES

XBW, BW, A1, A2, A3, A3.5, A4, B1, B2, CV, CVD, AO1, AO2, AO3, JE, AE

STORAGE

Recommended for optimal performance, store in a cool and dark place (4-25°C/39.2-77.0°F) away from high temperatures or direct sunlight.

PACKAGES

1. Syringes
1. Syringe 1.7g (1.0mL) x 1, Dispensing Tip Needle x 5, Dispensing Tip Long Needle x 5, Light-protective cap x 1
2. Syringe 1.7g (1.0mL) x 2, Dispensing Tip Needle x 10, Dispensing Tip Long Needle x 10, Light-protective cap x 2
3. Dispensing Tip Refill: Dispensing Tip Long Needle x 30, Light-protective cap x 2
- II. Unitips
1. Unitip 0.27g (0.16mL) x 15

CAUTION

1. In case of contact with oral tissue or skin, remove immediately with cotton or a sponge soaked in alcohol. Flush with water. To avoid contact, a rubber dam and/or COCOA BUTTER can be used to isolate the operation field from the skin or oral tissue.
2. In case of contact with eyes, flush immediately with water and seek medical attention if irritation persists.
3. Take care to avoid ingestion of the material.
4. Prior to starting the filling process, partially extrude paste outside of the patient's mouth and away from the patient to ensure paste is flowing properly.
5. Do not mix with other similar products.
6. Dispensing tips are for single use only. To prevent cross contamination between patients, do not reuse the dispensing tip. The dispensing tip cannot be sterilised in an autoclave or chemically.
7. For infection control reasons, Unitips are for single use only.
8. Wear plastic or rubber gloves during operation to avoid direct contact with inhibited resin layers in order to prevent possible sensitivity.
9. Wear protective eye glasses during light curing.
10. When polishing the polymerised material, use a dust collector and wear a dust mask to avoid inhalation of cutting dust.
11. Avoid getting material on clothing.
12. In case of contact with unintended areas of tooth or prosthetic appliances, remove with instrument, sponge or cotton pellet before light-curing.
13. Do not use G-aenial Universal Injectable in combination with eugenol containing materials as eugenol may hinder G-aenial Universal Injectable from setting.
14. Do not wipe the dispensing tip with excessive alcohol as this may cause contamination of the paste.
15. Dispose of all wastes according to local regulations.
16. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
17. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarise yourself with the safety data sheets available at: <http://www.gcamerica.com>

They can also be obtained from your supplier.

CLEANING AND DESINFECTING:
MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.

DO NOT IMMERSER. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level regulated healthcare-grade infection control product according to regional/national guidelines.

Last revised: 04/2019

Rx Only

MANUFACTURED BY
GC DENTAL PRODUCTS CORP.
2-285 Torimatsu-cho, Kasugai, Aichi 486-0844, Japan

RESPONSIBLE MANUFACTURER IN CANADA
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.

DISTRIBUTED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900
www.gcamerica.com

GC SOUTH AMERICA
Rua Heliodora, 399 - Santana - São Paulo, SP, BRASIL
CEP: 02022-051 - TEL: +55-11-2925-0965
CNPJ: 08.279.999/0001-61
RESF: TSC: Mayara de Santa Ribeiro - CRO/SP 105.982

GC ASIA DENTAL PTE. LTD
11 Tampines Concourse, #03-05, Singapore 528729
TEL: +65 6546 7588

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