




Prior to use, carefully read the instructions for use.

EN

IDENTIFICATION OF PARTS BEZEICHNUNG DER TEILE IDENTIFICATION DES PIÈCES I COMPONENTI DELL'IDRAULICO IDENTIFICACIÓN DE LAS PARTES

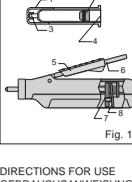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DIRECTIONS FOR USE GEBRAUCHSANWEISUNG MODE D'EMPLOI ISTRUZIONI PER L'USO INSTRUCCIONES DE USO

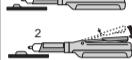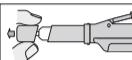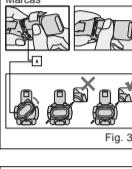GC Fuji ORTHO BAND PASTE PAK

GLASS IONOMER ORTHODONTIC BAND CEMENT

For use only by a dental professional in the recommended indications.

GC Fuji ORTHO BAND PASTE PAK is the first glass ionomer orthodontic band cement available in a cartridge pack and can be dispensed easily and correctly using a special dispenser called Paste Pak Dispenser.

RECOMMENDED INDICATIONS

1. Cementation of orthodontic bands.
2. Cementation of appliances retained with stainless steel crowns (e.g., Herbst or expansion appliances).

CONTRAINDICATIONS

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

IDENTIFICATION OF PARTS

(Paste Pak Cartridge)

1. Cover 2. Inner plunger 3. Cartridge tip

(Paste Pak Dispenser)

4. Piston 5. Adjusting guide 6. Lever

7. Sliding block 8. Piston release (Fig. 1)

DIRECTIONS FOR USE

Paste / Paste Ratio (g/g)	2.0 / 1.0
Mixing time (sec.)	15"-20"
Working time (min. sec.)	3'30"
Net setting time (min. sec.)	3'00"

Test conditions : Temperature (23 +/- 1°C)
Relative humidity (50 +/- 5%)

1. Paste Pak Cartridge Loading

a. Make sure that the piston is completely retracted into the dispenser. (Fig. 2)

Note : To retract the piston, hold both the sliding block and piston release (on the underside of the dispenser) with fingers and pull them back together.

b. Load the cartridge into the Paste Pak Dispenser. (Fig. 3)

1) Ensure the mark on the cartridge lines up with the mark on the Paste Pak Dispenser as illustrated.

2) Then turn the cartridge fully to the right position.

Note : To remove the cartridge, retract the piston fully and turn the cartridge back to the marked positions.

c. After cartridge loading, push the piston release forward until it stops. (Fig. 4)

1) Push the piston release forward.

2) Do not push the upper side.

2. Dispensing

a. Remove the cartridge cover. (Fig. 5)

b. Depress the lever to dispense the required amounts of pastes onto the mixing pad.

Note : 1) Before dispensing the first material from a new cartridge or when dispensing after a long interval, bleed the pastes in order to prevent the incorporation of air bubbles into the material.

2) The adjusting guide can be moved forwards or backwards to change the quantity of material to be dispensed. (Fig. 6)

1. Position of adjusting guide for a smaller quantity.

2. Position of adjusting guide for a larger quantity.

c. While depressing the lever, level and cut off the extruded material from the tips of the cartridge by moving the cartridge and dispenser to an upright position on the mixing pad. (Fig. 7)

1. Level and cut off material.

2. Depressing the lever.

When the lever is released after dispensing, residual pastes at the tips are retracted into the cartridge. Replace the cartridge cover.

Note : 1) If any surplus paste is retained on the cartridge tips after dispensing, remove with gauze, etc.

2) The amount of residual material in the cartridge can be confirmed by the position of the sliding block on the underside of the dispenser. As the cartridge has translucent tips, the residual amount can also be confirmed by the position of the inner plunger seen through the cartridge tips.

3) Do not drop the dispenser with the cartridge. Otherwise the cartridge may be damaged.

3. Mixing

After dispensing, incorporate and spread the material out in a thin layer on the mixing pad using a plastic spatula. Mix thoroughly, with lapping strokes, for 15-20 seconds. Be careful not to incorporate air bubbles.

Note : 1) The working time is 3 minutes 30 seconds from the start of mixing at 23°C (73.4°F). Higher temperatures will shorten working time. Beware that the cement will set sharp in the mouth.

2) Do not mix pastes with other materials.

4. Band preparation

a. After adapting the bands, remove any contaminants from the internal surfaces.

b. For improved retention, roughen the internal surfaces of the bands with a rotary instrument or by sandblasting or microetching.

5. Tooth cleaning

a. Clean the tooth surfaces to be cemented with pumice and water.

b. Rinse thoroughly. Blot away excess water with a cotton pellet or sponge. DO NOT DESICCATE. Enamel surfaces should appear moist (glistening).

6. Cementation technique

a. Bands

1) Using a suitable instrument, place the material throughout the internal surfaces of the bands.

2) Seal the bands.

3) When bands are in the final position, begin removing excess cement after initial set.

b. Appliances retained with stainless steel crowns

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