Prior to use, carefully read  
the instructions for use.

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ES

# NOGENOL™ Temporary Cement

FOR USE WITH TEMPORARY RESTORATIONS  
(NO EUGENOL)

For use only by a dentist or physician in the recommended indications.

## INSTRUCTIONS FOR USE

Personal Protective Equipment (PPE) such as gloves, face mask and safety eyewear should always be worn.

1. Extrude equal lengths of the accelerator and base pastes on a mixing pad. One gram accelerator to one gram base, or equal weight.
2. Spatulate the two pastes for approximately 30 seconds. Mixed paste will remain soft on the pad for a minimum of 3 minutes.
3. Spread a thin layer of the cement evenly over the dried inside surface of the restoration. NOGENOL temporary cement flows freely under the restoration in the mouth, and then sets rapidly.
4. Firmly seat restoration over the prepared tooth or teeth.
5. After the cement has set in 3-5 minutes, trim away excess.  
Note: This product is not intended to remain in the mouth for more than 30 days.

## IMPORTANT

If a faster set is desired, use more accelerator paste.

NOGENOL Temporary Cement should be stored in a cool and dark place (4-25°C; 39.2-77.0°F). If the Base tube should thicken or start to solidify, immerse the sealed Base tube in hot water (122°F = 50°C) for at least 5 minutes before extruding the material for use.

## STORAGE

Recommended for optimal performance, store in a cool and dark place (4-25°C; 39.2-77.0°F).

Shelf-life: 3 years from date of manufacture.

## PACKAGES

### 136421 NOGENOL Temporary Cement Base

Contains: One refill (30g).

### 136411 NOGENOL Temporary Cement Catalyst

Contains: One refill (65g).

### 136402 NOGENOL Temporary Cement Standard Package

Contains: One catalyst (65g) and 2 base (30g).

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the Safety Data Sheets available for the Americas at:

<http://www.gcamerica.com/downloads>

Last Revised: 06/2016

MANUFACTURED & DISTRIBUTED BY  
GC AMERICA INC.  
3737 West 127th Street  
Aurora, IL 60003 U.S.A.  
TEL: +1-708-597-0900  
[www.gcamerica.com](http://www.gcamerica.com)

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