Prior to use, carefully read the instructions for use.

## GC Fuji COAT LC

LIGHT CURED PROTECTIVE COATING

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS

For the prevention of water and saliva contamination to exposed surfaces of glass ionomer cements during the first 24 hours after placement.

### CONTRAINDICATIONS

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

### DIRECTIONS FOR USE

- a) Place the glass ionomer cement according to the manufacturer's instructions.
- b) Using a cotton pellet or brush apply GC Fuji COAT LC to all exposed glass ionomer and adjacent surfaces.
- c) Light cure for 10 seconds using suitable visible light curing device (>500mW/cm<sup>2</sup>).
- d) Close bottle immediately after use.

### STORAGE

Recommended for optimal performance, store in a cool and dark place (4-25°C) (39,2-77,0°F). (Shelf life : 2 years from date of manufacture).

### PACKAGE

Bottle of 5g (5.2mL)

### CAUTION

1. GC Fuji COAT LC is FLAMMABLE. Avoid sources of ignition and do not store in direct sunlight.
2. In case of contact with oral tissue or skin, flush immediately with water.
3. In case of contact with eyes, flush immediately with water and seek medical attention.
4. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:  
<http://www.gceurope.com>  
or for The Americas  
<http://www.gcamerica.com>  
They can also be obtained from your supplier.

### CLEANING AND DISINFECTING:

**MULTI-USE DELIVERY SYSTEMS:** to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged. **DO NOT IMMERSE.** Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Last revised : 11/2016

**Rx Only**

**GC**

MANUFACTURED by  
**GC CORPORATION**  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

RESPONSIBLE MANUFACTURER IN CANADA:

**GC AMERICA INC.**  
3737 West 127th Street, Alsip, IL 60803 U.S.A.

DISTRIBUTED by  
**GC CORPORATION**  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

**EU: GC EUROPE N.V.**  
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,  
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

**GC AMERICA INC.**  
3737 West 127th Street, Alsip, IL 60803 U.S.A.  
TEL: +1-708-597-0900  
[www.gcamerica.com](http://www.gcamerica.com)

**GC ASIA DENTAL PTE. LTD.**  
11 Tampines Concourse, #03-05 Singapore 528729  
TEL: +65 6546 7588

**GC AUSTRALASIA DENTAL PTY. LTD.**  
1753 Botany Rd, Banksmeadow, NSW 2019, Australia  
TEL: +61 2 9301 8200

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