Prior to use, carefully read the instructions for use.

**EN**

## GC Fuji VARNISH

PROTECTIVE COATING

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS

For the prevention of water and saliva contamination to exposed surfaces of glass ionomer cements during the first 24 hours after placement.

### CONTRAINDICATIONS

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

### DIRECTIONS FOR USE

- a) Place the glass ionomer cement according to the manufacturer's instructions.
- b) Using a cotton pellet or brush apply GC Fuji VARNISH to all exposed glass ionomer and adjacent surfaces.
- c) Dry by gently blowing with air syringe.
- d) Maintain moisture isolation for 2-3 minutes.

Note :
GC Fuji VARNISH is volatile. Close bottle immediately after use.

### STORAGE

Store at temperature of 4-25°C (39.2-77.0°F). (Shelf life: 3 years from date of manufacture)

### PACKAGE

Bottle of 10g (10.4mL)

### CAUTION

1. GC Fuji VARNISH is FLAMMABLE. Avoid sources of ignition and do not store in direct sunlight.
2. In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
3. In case of contact with eyes, flush immediately with water and seek medical attention.

Last revised : 01/2009

MANUFACTURED by  
GC ASahi CORP.  
1-3-3 Kachigawa-cho, Kasugai, Aichi 486-0945, Japan

DISTRIBUTED by  
GC CORPORATION  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

 EU : GC EUROPE N.V.  
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,  
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

GC AMERICA INC.  
3737 West 127th Street, Alsip, IL 60803 U.S.A.  
TEL: +1-708-597-0900.

GC ASIA DENTAL PTE. LTD.  
11 Tampines Concourse, #03-05, Singapore 528729  
TEL: +65 6546 7588

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