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Prior to use, carefully read the instructions for use.

EN

GC Pas/Cion

ALGINATE IMPRESSION MATERIAL

For use only by a dental professional.

INDICATIONS:

Preliminary impressions.
Impressions for fabrication of:
- Models for general dental practice.
- Orthodontic models
- Study models

DIRECTIONS FOR USE

Mixing time	20-30 sec.
Total working time	70 sec.
Minimum time in mouth	60 sec.
Initial setting time	110 sec.
Final setting time	130 sec.

1. Mix the following amount of powder and water: 10g of powder for 20mL of water (Fig. 1).
2. Mix for 20-30 seconds. Spread mixed material evenly on the impression tray (Fig. 2).
3. Carefully fit the tray in the mouth. Refer to table above for recommended Total working times.
4. Wait until material sets. Refer to table above for the minimum time in mouth (Fig. 3).
5. Remove tray from the mouth. Rinse the impression under running tap water (Fig. 4).

Note:
1. Final setting time includes mixing time and minimum manipulation time. This is based on mixing at 23°C with distilled or deionized water.
2. Higher temperatures of mixing water will shorten the setting and total working time. Variations in hardness of the water may influence setting time.

DISINFECTION

The recommended disinfection method is soaking in a 0.1% sodium hypochlorite solution for 15 minutes.
Refer to the respective manufacturer's instructions for use.

MODEL FABRICATION

1. Remove excess water before pouring cast. For most accurate result, pour cast immediately after rinsing (Fig. 5).
2. If the impression cannot be poured immediately, wrap the impression in a wet paper towel or store the wet impression in a tightly sealed plastic bag.
3. DO NOT STORE IN WATER.
4. GC Pas/Cion is compatible with type III and type IV stone (GC "NEW FUJIROCK", GC FUJIROCK™ EP).

STORAGE

Recommended for optimal performance, store in a dry location at room temperature.
(Shelf life: 3 years from date of manufacture)

CAUTION:

1. Product may cause sensitivity in some people in rare cases. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
2. In case of contact from material or mixture with eyes, flush them immediately with water and seek medical attention.
3. When using material, wear a dust mask to avoid inhaling powder dust.
4. After opening, transfer the contents of the package into a container designed for exclusive use of these materials (recommended to use GC "CHAPPY" - <http://www.gcaustralasia.com/chappy.html>).
5. Do not mix with any other product.
6. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn while working or manipulating this material.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gcamerica.com>
They can also be obtained from your dealer and suppliers.

Last revised : 11/2016

GC

MANUFACTURED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED BY
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900
www.gcamerica.com

GC South America
Rua Heliódora 399, Santana - 02022-051
São Paulo, SP Brasil
Tel: +55-11-2925-0965
CNPJ: 08.279.999/0001-61
Responsável Técnica
Mayara de Santis Ribeiro
CRO/SP 105.982