# MI Varnish®

Topical Fluoride Varnish with Calcium and Phosphate

For use only by a dental professional in the recommended indications.

## DESCRIPTION

MI Varnish is a 5% sodium fluoride varnish that has a desensitizing action when applied to tooth surfaces. MI Varnish also contains RECALDENT™\* (CPP-ACP): Casein Phosphopeptide-Amorphous Calcium Phosphate. The application leaves a film of varnish on tooth surfaces.

## INDICATIONS

Treatment of hypersensitive teeth.

## CONTRAINDICATIONS

1. Do not use on patients with ulcerative gingivitis and stomatitis.
2. Casein Phosphopeptides are derived from milk casein. Do not use this material on patients with a proven or suspected milk protein allergy.

## PRECAUTIONS

1. In rare cases, the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
2. When MI Varnish is applied, patients should not use other prescriptive fluoride treatment such as fluoride gels during the same day. The routine use of fluoride tablets by patients should be interrupted for several days after treatment.

## DIRECTIONS FOR USE

1. Tooth surfaces should be cleaned and dried before application of MI Varnish. A prophylaxis is not required.
2. Peel off the foil lid of the unit dose container of MI Varnish.
3. Apply a thin, uniform layer of MI Varnish on teeth / surfaces using a disposable brush.  
Note:  
If separation is observed, stir with the disposable brush before application.
4. MI Varnish sets when in contact with water or saliva.
5. MI Varnish should remain undisturbed on the teeth for four hours. Instruct patients to avoid hard, hot or sticky foods, tooth brushing and flossing, products containing alcohol (oral rinses, beverages, etc.) during this time period.

## ADVERSE REACTIONS

Edematous swelling has been reported only in rare instances in some fluoride varnish products, especially after application to extensive surfaces. Although extremely rare, dyspnea has occurred in asthmatic children. Nausea has been reported when extensive applications have been made to patients with sensitive digestive systems.

If required, varnish film can be removed by thoroughly brushing once set.

## FLAVOR

Fresh Strawberry  
Fresh Mint

## STORAGE

Recommended for optimal performance, store in a cool, dry and low humidity area away from direct sunlight (8-25°C) (46.4-77.0°F).  
(Shelf life: three years from date of manufacture)

## PACKAGE

One unit dose package contains 0.44g/0.4mL  
50 unit dose packages  
50 disposable brushes

Note:  
1 mL of MI Varnish contains 50 mg of sodium fluoride (22.6 mg fluoride ion).

## CAUTION

1. Avoid contact with soft oral tissues, eyes and skin because MI Varnish contains ingredients that may be irritating to them. In case of contact with soft oral tissue, remove by gentle brushing. In case of contact with eyes, flush immediately with water and seek medical attention. In case of contact with skin, wash immediately with soap and water.
2. MI Varnish is flammable. Do not use near open flame. Keep away from sources of ignition.  
Do not store large quantities in one area. Keep away from direct sunlight.
3. Avoid swallowing or digestion of material.
4. MI Varnish unit dose and brush are disposable. Do not reuse.
5. In case of contact with clothes, table or floor, immediately wipe off with ethanol.
6. Do not mix with other products.
7. Dispose of all waste according to local regulations.
8. MI Paste Plus® should not be used on the same day but can be used from the following day. MI Paste® (without Fluoride) can be used four hours after application of MI Varnish.
9. Personal Protective Equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

## NOTE

RECALDENT™ (CPP-ACP) Technology was developed by School of Dental Science, The University of Melbourne, Australia.

\*RECALDENT is a trademark used under license.

RECALDENT™ (CPP-ACP) Technology has related patents or patents pending in Australia, NZ, Europe, Canada and USA.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available for the Americas at: <http://www.gcamerica.com>

They can also be obtained from your supplier.

Last Revised: 08/2017

**GC**

MANUFACTURED by  
GC CORPORATION  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED by  
GC CORPORATION  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan  
GC AMERICA INC.  
3737 West 127th Street, Alsip, IL 60803 U.S.A.  
TEL: +1-708-597-0900  
[www.gcamerica.com](http://www.gcamerica.com)

GC SOUTH AMERICA  
Rua Heliodora, 399, Santiana - São Paulo, Sp - Brasil  
CEP: 02022-051 - TEL: +55-11-2925-0965  
CNPJ: 08.279.999/0001-61  
RESP. TEC: Mayara De Santis Ribeiro - CRO/SP: 105.982

PRINTED IN THE U.S.A.

Prior to use, carefully read the instructions for use.

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