GC RELINE™ POINT FOR TRIMMING GC RELINE™ WHEEL FOR FINISHING

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

1. GC RELINE™ POINT FOR TRIMMING: Trimming the edge of GC RELINE™ Soft or Extra Soft.
2. GC RELINE™ WHEEL FOR FINISHING: Finishing the trimmed edge of GC RELINE™ Soft or Extra Soft.

DIRECTIONS FOR USE

1. Take a sharp knife and remove excess material on the edge of the GC RELINE™ Soft or Extra Soft.
2. Fit the GC RELINE™ POINT FOR TRIMMING or WHEEL FOR FINISHING into the handpiece. The wheel must be beforehand set on a mandrel with a fixation cap.
3. Using light pressure, progressively trim the edge with the point using a speed of less than 20,000 RPM and finish with the wheel using a speed of less than 6,000 RPM.

NOTES

1. Before using, check that the shaft of the GC RELINE™ POINT FOR TRIMMING or WHEEL FOR FINISHING rotates correctly without distortion.
2. Rotate the point or wheel from the relined material toward the denture base.
3. Do not use excessive pressure when working with the point or wheel.
4. When trimming or finishing the GC RELINE™ Soft or Extra Soft material, wear a dust mask to avoid inhaling silicone dust, and wear protective eye glasses.

PACKAGES

1. GC RELINE™ POINT FOR TRIMMING : 3 pieces
2. GC RELINE™ WHEEL FOR FINISHING : 12 pieces

GC RELINE™ POINT FOR TRIMMING GC RELINE™ WHEEL FOR FINISHING

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3737 West 127th Street, Alsip, IL 60803 U.S.A.
GC ASIA DENTAL PTE. LTD.