Prior to use, carefully read the instructions for use. **EN**

everX Posterior™

FIBER REINFORCED COMPOSITE FOR DENTIN REPLACEMENT

For use only by a dental professional in the indications for use.

INDICATIONS FOR USE

everX Posterior is suitable for use as the reinforcing material for direct composite restorations, especially in large posterior cavities, for example:

- Cavities including 3 surfaces or more
- Cavities with missing cusps
- Deep cavities (including class I, II and endodontically treated teeth)
- Cavities after amalgam replacement
- Cavities where onlays & inlays would also be indicated

Note: everX Posterior should always be covered with a layer of light-cured universal restorative composite, for sufficient wear resistance

CONTRAINDICATIONS

- Direct pulp capping
- Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.
- At least one horizontal dimension of the cavity should exceed 3 mm.
- The product is not suitable for patients with a history of hypersensitivity to methacrylate monomers.
- Do not use as final approximal and surface composite layer.

PRODUCT DESCRIPTION

everX Posterior is a light-cure, radio-opaque restorative material to be used intra-orally and classified as a Type 2 and Class 2 (Group 1) per ISO standard 4049. This material has a radiopacity equivalent to 2.5-3.0 mm of aluminium (dentin = 1 mm, enamel = 2 mm). The particle size of inorganic fillers range is 0.02-0.7 µm. The average length of fibres is 800 µm and the diameter is 17 µm. The total amount of inorganic filler is approximately 57 vol%.

COMPOSITION

Barium glass, dimethacrylate, glass filler, silicon dioxide, initiator, methacrylate polymer, pigment, stabilizer

DIRECTIONS FOR USE

- Shade Selection Clean the tooth with pumice and water. Shade selection of the final composite enamel layer should be made prior to isolation.
- Cavity Preparation (Fig. 1) Prepare cavity using standard techniques. Dry by gently blowing with oil free air. The use of rubber dam is recommended to isolate prepared tooth from contamination with saliva, blood or saliva fluid. Note: For pulp capping, use calcium hydroxide.
- Bonding Treatment (Fig. 2) For bonding everX Posterior to enamel and/or dentin, use a light-cured bonding system such as G-Premio BOND, G-anial Bond or G-BOND. Follow manufacturer's instructions.
- Application of everX Posterior (Fig. 3) – see note for Class I and large cavities Insert everX Posterior into the Unitip APPLIER 1 or equivalent. Remove the cap, and extrude everX Posterior material directly into the cavity. Pack the everX Posterior in the cavity, using suitable instruments, until it has been adapted tightly against the cavity floor and walls. To prevent premature curing, avoid bright light in the work area. When placing everX Posterior material in the cavity, ensure that there is enough space for the overlying composite on all surfaces of the restoration (refer to point 6).

- Use a suitable matrix system to establish the proximal contour and contact area.
- When placing everX Posterior in Class 3 or larger cavity, build first the missing walls by using light-curing restorative composite before placing everX Posterior material. The wall should be thick enough to withstand the application pressure of everX Posterior. Note: Material may be hard to extrude immediately after removing hot oil storage. Prior to use, leave to stand for a few minutes at normal room temperature. Minimize exposure to ambient light. Ambient light can shorten the manipulation time.

Clinical Hint

Optionally, apply first a thin layer of flowable composite (e.g., G-anial Flo X GRADIA DIRECT Flo) to the cavity floor before the application of everX Posterior. Place the everX Posterior on top of the flowable composite and pack it in the cavity.

Light Curing (Fig. 4)

Light cure everX Posterior using a light curing unit. Keep light guide as close as possible to the composite. Refer to the following chart for irradiation Time and Effective Depth of Cure.

Irradiation Time and Effective Depth of Cure			
Curing device		Irradiation Time	Depth of cure
High Power LED Light (>1200 mW/cm ²)		10 sec.	4 mm
Halogen/ Normal LED Light (>700 mW/cm ²)		20 sec.	4 mm
The effective wavelength range of each dental curing unit must cover 450–480 nm.			
Note: Lower light intensity may cause insufficient curing or discoloration of the material.			

- Placement of the overlying composite (Fig. 5) Fill the remaining cavity space with a wear-resistant and polishable restorative composite, such as G-anial Posterior or GC KALORE. The composite layer should be of 1-2 mm on the occlusal surface. Light-cure as instructed by the manufacturer.
- Finishing and Polishing Check and adjust the occlusion. Finish and polish using standard techniques.

SHADE

everX Posterior is available in one universal shade.

STORAGE

Recommended for optimal performance, store in a cool and dark place (4-25°C / 39-77°F).

PACKAGES

15 units (0.25 g / 0.13 mL)

50 units (0.25 g / 0.13 mL)

CAUTION

- In case of contact with oral tissue or skin, remove immediately with cotton or a sponge soaked in alcohol. Flush with water. To avoid contact, a rubber dam and/or COCOA BUTTER can be used to isolate the operation field from the skin or oral tissue.
- In case of contact with eyes, flush immediately with water and seek medical attention to avoid ingestion of the material.
- Prior to starting the filling process, partially extrude paste outside of the patient's mouth and away from the patient to ensure paste is flowing properly. Do not mix with other similar products.
- For infection control reasons, Units are for single use only.
- Wear rubber-free plastic or rubber gloves during handling. Avoid direct contact with air inhibited resin layers in order to prevent possible sensitivity.
- Wear protective eye glasses during light curing.
- When polishing the polymerized material, use a dust collector and wear a dust mask to avoid inhalation of curing dust.
- Avoid getting material on clothing.
- In case of contact with unintended areas of tooth or prosthetic appliances, remove with instrument, sponge or cotton pellet before light curing.
- Do not use everX Posterior in combination with eugenol containing materials as eugenol may hinder everX Posterior from setting.
- Do not wipe the unitip with excessive alcohol as this may cause contamination of the paste.
- Dispose of all wastes according to local regulations.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at <https://www.gc.dental/europe>. They can also be obtained from your supplier.

CLEANING AND DESINFECTING: MULTI-USE DELIVERY SYSTEMS: To avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged. **DO NOT IMMERSIE.** Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

Undesired effects: Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority or the country accessibility from the following links: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental. In this way you will contribute to improve the safety of this product.

UK Responsible Person GC UNITED KINGDOM Ltd. Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JL, United Kingdom Last revised: 06/2025

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