## LINING CEMENT

RADIOPAQUE GLASS IONOMER LINING CEMENT

Prior to use, carefully read the instructions for use.

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS

As a base or liner in prepared cavities.

### CONTRAINDICATIONS

- Pulp capping.
- In rare cases the product may cause sensitivity to some persons. If such reactions are experienced, discontinue the use of the product and refer to a physician.

### DIRECTIONS FOR USE

#### 1. POWDER AND LIQUID DISPENSING

- The standard powder to liquid ratio is 1.2/1.0g. 1 level scoop of powder to 1 drop of liquid.  
Note :  
For a base, a heavier consistency is desirable. Compress powder in the scoop before levelling.
- For accurate dispensing of powder, lightly tap the bottle against the hand. Do not shake or invert.
- Hold the liquid bottle vertically and squeeze gently.
- Close bottles immediately after use.

#### 2. MIXING

Dispense powder and liquid onto the pad. Using the plastic spatula, divide the powder into two equal parts. Mix the first portion with all of liquid for 5-10 seconds. Incorporate the remaining portion and mix the whole thoroughly for 10-15 seconds.

#### 3. RESTORATIVE TECHNIQUE

- Prepare tooth using standard techniques. Extensive mechanical retention is not necessary. For pulp capping use calcium hydroxide.
- Wash and dry but **DO NOT DESICCATE**. GC CAVITY CONDITIONER or GC DENTIN CONDITIONER is recommended to remove the smear layer.
- Mix the required amount of cement.
- Remove surface moisture, but **DO NOT DESICCATE**.
- Transfer cement to the preparation using a syringe or suitable placement instrument, covering dentine up to the dentino-enamel junction.
- Allow cement to set (about 4 minutes).
- Proceed with standard composite or amalgam restorative techniques.

### STORAGE

Store at temperature of 4-25°C (39.2-77.0°F).  
(Shelf life : 3 years)

### PACKAGES

- Bottle of 15g powder with measure.
- Bottle of 15g (12 mL) liquid.
- 1-1 pkg. : 15g powder, 15g (12 mL) liquid, powder spoon, plastic spatula, mixing pads.

### CAUTION

- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- DO NOT** mix powder or liquid with any other glass ionomer components.

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