## EXA'lence™ PUTTY

Prior to use, carefully read the instructions for use.

VINYL POLYSILOXANE IMPRESSION MATERIAL

For use only by a dental professional in the recommended indications.

Indication	Impression Technique	Tray Type	Recommended Viscosities
Impressions of facing, inlay, onlay, crown or bridge	Double mix double impression (Putty - wash impression)	Stock or custom tray	Putty + Medium Body or (Extra) Light Body

### CONTRAINDICATIONS

In rare cases, the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

#### PHYSICAL PROPERTIES (Average)

	EXA'lence PUTTY (Type 0: very high consistency)
Mixing time	20 sec.
Total Working Time (Minutes)	1 minute
Minimum time in mouth (Minutes)	3 minutes
Recovery from deformation	99,2%
Maximum strain in compression	1,4%
Linear dimensional change	≤ - .10 <span> </span> %
Detail reproduction	75µm
Compatibility with gypsum	75µm

Test conditions: Temperature (74°F/23°C ± 4°F/2°C) Relative humidity (50 ± 5%) (ISO 4823: 2000 E) (Dental elastomeric impression materials)

### PRECAUTIONS

- When mixing EXA'lence PUTTY, care should be taken to avoid mixing or contact with the following materials. These may delay or prevent setting: Catalyst for condensation silicone impression materials, polysulfide Impression Materials, eugenol materials, Sulfur, Latex, Oil, Acrylates, Glycerol, Spray-type or ointment-type topical surface anesthesia (e.g. lidocaine).
- When kneading material, put on plastic gloves as they polyethylene gloves. Do not use latex gloves as it may delay setting and cause rough surfaces.
- Do not change container caps and measuring spoons.
- The putty impression should be cleaned and dried completely before taking final impression.

#### DIRECTIONS FOR USE

Note:

Prior to use, leave the material to stand at room temperature. Cooled or refrigerated material may delay setting.

- Make sure the preparation is dry and free of debris during impression taking.
- Apply a tray adhesive (GC Universal VPS Adhesive, etc.) onto the tray and leave to dry. Read manufacturer's instructions for use.
- Take equal amounts of Base and Catalyst and knead for 20 seconds until the color of mixture is uniform.
- Fill the tray with the mixed material and insert it into mouth using a spacer within 1 minute from start of mix.
- Wait for 3 minutes to set and remove the impression from the mouth. Remove the spacer.
- Rinse and dry the impression. Make sure the impression is clean and dry prior to taking final impression.
- Syringe the selected wash material onto the prepared tooth and the preliminary impression.

Note:

Blasting with air is not necessary. EXA'lence wash materials' hydrophilic property enables better selfwetting, allowing the excellent adaptation to the tooth.

- Seat the loaded tray in the mouth.
- Wait for set. Refer to the working times and setting times of the selected wash material in the respective instructions for use.
  - EXA'lence's cartridge material has unique chemistry that allows it to polymerize faster in the oral cavity than at room temperature. Therefore to determine setting, examine material in the mouth since material on bench will yield a false result.
  - Higher temperatures shorten working time and lower temperatures increase working time.
- Remove impression and rinse under running tap water.

#### DISINFECTION

The impression can be disinfected with a glutaraldehyde solution or other appropriate disinfectants. Refer to the respective manufacturer's instructions for use.

#### MODEL FABRICATION

- After the impression is rinsed with water, wait for 60 minutes to dry the impression before pouring a stone model.
- Pour a model within 14 days after impression taking. Impression is compatible with type III and type IV stone (GC Fujirock™ EP) and can be silver or copper plated.

#### CAUTION

- Avoid contact of base, catalyst, mixed material or adhesive with eyes. In case of contact, immediately flush with generous amount of water and seek medical attention.
- Avoid getting material on clothing as it is hard to remove.
- Do not mix with other products.

#### STORAGE

Store in a cool and dry place at room temperature (59 - 77°F; 15 -25°C) (Relative humidity: 50 +/- 10%). Avoid direct sunlight. Shelf-life: 2 years from date of manufacture.

#### PACKAGES

Kit contains:

Base Putty 500g (278mL) .....1
Catalyst Putty 500g (278mL) .....1
Measuring spoon for base .....1
Measuring spoon for Catalyst .....1
Exa'lence Putty Clinic Package

Last revised: 06/2010

**'GC'** **CE**  
DISTRIBUTED BY  
GC AMERICA INC.  
3737 West 127th Street, Alsip, IL 60803, U.S.A.

**EU:** GC EUROPE N.V.  
Interleuvenlaan 33, 3001 Leuven, Belgium  
GC ASIA DENTAL PTE. LTD.  
19 Loyang Way, #06-27 Singapore 508724

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