Prior to use, carefully read
the instructions for use.

EN

EPITEX™

FINISHING AND POLISHING STRIPS

Aluminum Oxide abrasives

For use only by a dental professional in the recommended indications.

INTENDED USE

For contouring, finishing and polishing of interproximal composite, compomer and glass ionomer restorative surfaces and prophylactic cleaning.

| Width | Length | Thickness | Grit | Use |
|----------------------------|--------|-----------|-------|----------------------------------|
| Coarse (Blue) | 5 mm | 10 m | 75 µm | #280 Contouring |
| Medium (Green) | 5 mm | 10 m | 50 µm | #500 Finishing |
| Fine (Grey) | 5 mm | 10 m | 45 µm | #800 Polishing |
| Extra Fine (Pink) | 5 mm | 10 m | 40 µm | #1200 Polishing |
| Matrix strip (Transparent) | 8 mm | 10 m | 50 µm | - Contouring prior to light cure |

1. Remove excess material with a suitable bur.
2. Choose a suitable grit.
3. Insert the strip between the interproximal contact point. Firmly grab both ends of the strip and draw back and forth.

4. Discard used strips.

5. If any of these products may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

6. EPITEX has sharp edges. Wear protective gloves and eyewear when handling the strips.

Take care not to injure the patient's soft tissues.

7. For infection control reasons, strips are for single use only. Dispose of all waste according to local regulations.

8. Personal protective equipment (PPE) such as gloves, masks and safety eyewear should always be worn.

PACKAGES

- 1 Starter Kit
- 1 reel (10 m) of each grit and Matrix Strip, 1 piece of dispenser stand
- 2 Refill pack available in all grits and Matrix strip 1 reel (10 m)

STORAGE

Recommended for optimal performance, store away from high temperature or humidity.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

<http://www.gcamerica.com>

They can also be obtained from your supplier.

Undesired effects- Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

https://ec.europa.eu/growth/sectors/medical-devices/contacts_en

as well as to our internal vigilance system:

vigilance@gc.dental

In this way you will contribute to improve the safety of this product.

UK Responsible Person

GC UNITED KINGDOM Ltd.

Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JS United Kingdom

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