



COECIDE™ XL PLUS

STERILIZING AND DISINFECTING SOLUTION

PRODUCT DESCRIPTION



COECIDE XL PLUS, a 3.4% glutaraldehyde, has broad spectrum activity that is sporicidal, tuberculocidal, virucidal, pseudomonacidal and fungicidal. This 3.4% alkaline glutaraldehyde solution is designed to provide an "extra measure" of protection where hyperdilution and added bioburden are of concern. Shipped in unactivated gallons with a buffer solution for activation, COECIDE XL PLUS can be activated one quart at a time for economy of use.

FEATURES AND BENEFITS

Packaged in metered one gallon bottle with activator

- Total gallon does not have to be activated
- Bottle and activator marked for individual quart activation
- Convenient and economical

Disinfects immersed instruments at high levels in 90 minutes @ 25°C. Sterilize them in 10 hours at 25°C. COECIDE XL PLUS has a broad spectrum of activity which includes sporicidal, tuberculocidal, virucidal, bactericidal, pseudomonacidal and fungicidal when used as directed.

- Disinfection reduces possibility of cross-contamination

Contains corrosion inhibitors

- Protects valuable instruments from discoloration and corrosion

Special formula has wetting and penetration ability

- Solution reaches even the most inaccessible areas of instruments

Approved for reuse

- Can be used and reused up to 28 days after activation

PRODUCT CODES

550128 **COECIDE XL PLUS Gallon (3.89 L) Bottle**



GC America Inc.

Advancing the Art and Science of Dentistry

© 2001 GC America Inc., 3737 West 127th St., Alsip, Illinois 60803